

AnsuPen™

Reusable Insulin Pen

To use with Insulin Cartridge




User guideline



Thank you for choosing **AnsuPen™**, the intuitive Swiss insulin pen. This manual contains instructions for using, storing and cleaning your **AnsuPen™**.

Your **AnsuPen™** can deliver doses from 1 to 60 units, in increments of 1 unit. It is designed to be used with 3ml **Ansulim™** pen cartridges.

Description of parts of AnsuPen™

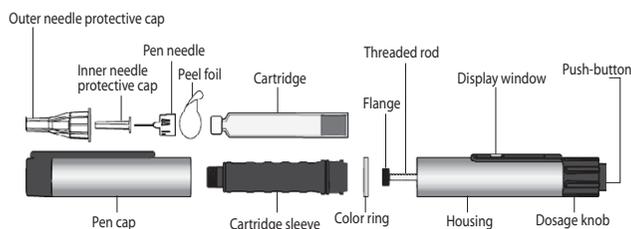


Pen Preparation / Changing Cartridge

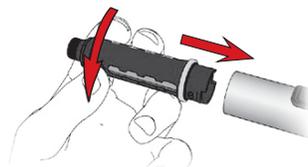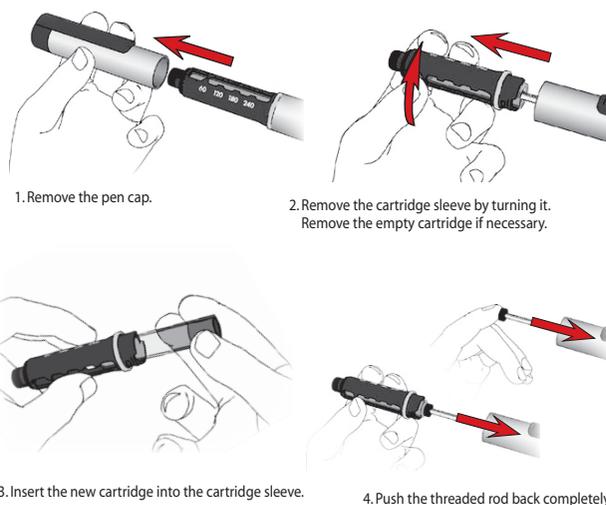

5. Attach the cartridge sleeve to the housing by turning it.


Attach Pen Needle

Follow the instructions for use of your pen needle.

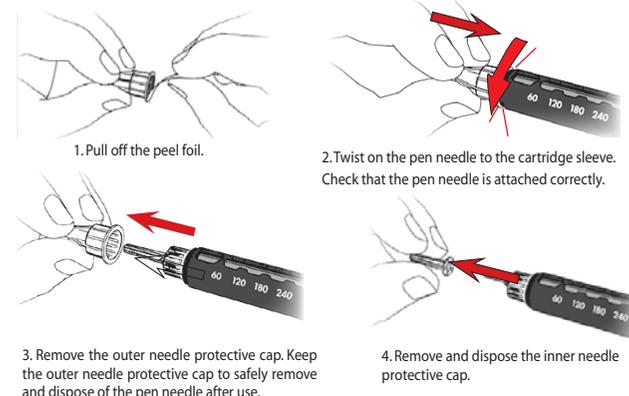

Priming / Function Check

With the function check any air remaining in the cartridge is removed and it is checked that the pen needle is not clogged.

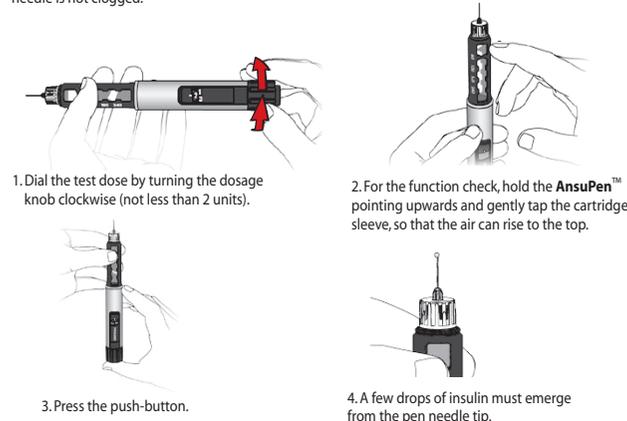


Page-1

Page-2

Page-3

Dose Dialing

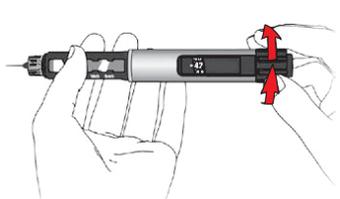

Dial the required insulin dose


Dial the required insulin dose by turning the dosage knob clockwise. Correct the dose by turning counter clockwise. Check the dialed dose by looking vertically from above and not at an angle onto the display, so that the numbers are clearly identified.

Safety stop

The insulin dose that can be set on the **AnsuPen™** is limited to the amount remaining in the cartridge. If the amount of insulin left in the cartridge is no longer sufficient for the desired insulin dose, the dosage knob cannot be turned any further in a clockwise direction.

Do not try to force the dosage knob to turn further. Note the remaining insulin dose that can be dispensed and inject. Change the cartridge and pen needle, and carry out function checks. Dial and inject the outstanding insulin dose.

Injection

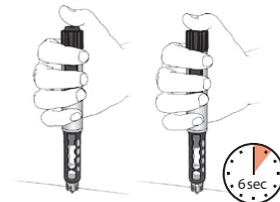

Use the injection technique as recommended by physician and/or health care professional. Press the push-button in fully for injection. Hold the push-button pressed down during medication discharge.

After the medication has been completely discharged, wait for 6 seconds and then pull out the pen needle slowly. You may either hold the push-button pressed or release it during the 6 seconds.



Remove pen needle after each injection

Carefully attach the outer needle protective cap on the pen needle.

Twist off the pen needle and dispose it correctly, according to the advice of your physician and/or health care professional.



Attach pen cap



Attach pen cap after each use.

Storage and care

Please store and handle your needles and cartridges according to the respective instructions for use. After each use, attach the pen cap. Always keep your **AnsuPen™** in the pen pouch, without a pen needle attached.

Use and store your **AnsuPen™** at room temperature below or equal 30°C. please handle your **AnsuPen™** with care. Protect it from water, dust and moisture. A damp cloth is sufficient to clean your **AnsuPen™**. Please do not use alcohol, other solvents or cleaning agents. Never immerse your **AnsuPen™** in water, as this could damage the pen.

Life Time

Life time of **AnsuPen™** is 3 years from first use.

For further information please contact: 01713067275

Made by:
YPSOMED AG
Burgdorf, Switzerland
www.ypsomed.com


For:

SQUARE
PHARMACEUTICALS LTD.
BANGLADESH

TM - Trade Mark.

2000006696

AnsuPen™ is designed to be used with Ansulim™ Pen Cartridges

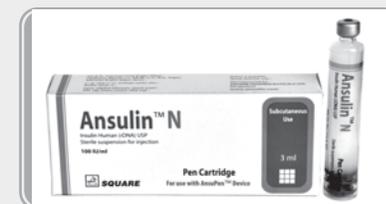

Page-1

Page-2

Page-3